



The AVIVO™ Mobile Patient Management (MPM) System *Indications for Use, Contraindications, and Precautions*

Rx only

Indications for Use

The AVIVO Mobile Patient Management System is intended to continuously record, store, and periodically transmit physiological data. The System is indicated for those patients who require monitoring for the detection of non-lethal cardiac arrhythmias. The AVIVO Mobile Patient Management System also monitors, derives and displays:

- ECG
- Heart Rate (including HR variability)
- Activity
- Posture
- Respiration rate (including RR variability)
- Body fluid status

Contraindications

- PiiX should not be used on patients with known allergies or hypersensitivities to adhesives or hydrogel.

Precautions

- PiiX should not be used on patients with implantable devices with active minute ventilation sensors.
- PiiX may cause mild discomfort, skin irritation, redness, itching, rash, contact dermatitis in some individuals. The device should be removed if any pain or discomfort occurs. If skin irritation or redness persists after the device has been removed, a topical anti-inflammatory cream may be applied to the area (in consultation with your health care provider).
- PiiX is intended for single patient use.
- PiiX should be removed prior to external defibrillation or an MRI scan.
- PiiX should not be applied to broken, damaged or irritated skin.
- PiiX is water resistant but not waterproof. It should not be submerged in water (showering is acceptable, but swimming and submersion bathing are prohibited).
- PiiX should not be disassembled.
- Replace the PiiX if it peels off; do not reapply the PiiX.
- Replace the PiiX if it appears damaged.
- No creams or lotions should be applied immediately prior to use of PiiX.
- This product is recommended for adult use.

The AVIVO Patient Management System is not intended to replace direct communication with your healthcare provider. The system data should not be used alone, but should be used along with all other clinical data and exams to come to a diagnosis. Additionally, it will not summon emergency response personnel in the event you need help. Talk to your healthcare provider immediately if you have any concerns, or if your condition changes. The AVIVO System is not an emergency response service. Seek medical advice if you experience any symptoms that concern you.

Corventis, Inc.
2033 Gateway Place, Suite 100
San Jose, CA 95110
U.S.A.

Corventis, Inc.
1410 Energy Park Drive, Suite 1
St. Paul, MN 55108
U. S. A.

UAE Authorized Distributor
Xenith, LLC
PO Box 29602
103, UB Building
Garhood
Dubai, UAE

**EU Authorized
Representative**
Corventis Europe BVBA
Ikaroslaan 19
1930 Zaventem
Belgium

Singapore
Corventis PTE. LTD.
#05-01 The Franklin
3 Science Park Drive
Singapore Science Park 1
Singapore, 118223

Customer Service Numbers:
USA: 1 (877) 247 PiiX (7449)
1 (408) 790-9393
EU: +32-2-720 21 67
India: +91-44-30922512
Singapore: +1-800-888-PiiX (7449)
All Other: +1 (408) 790-9399

